

# Data Upload

1. Connect the Bravo® recorder to your PC using the supplied **USB cable**. (Fig. 1)



Fig. 1

2. Open AccuView™ by double-clicking the AccuView icon on your desktop. (Fig. 2)

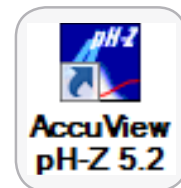


Fig. 2

3. Click **Upload**. A progress bar appears. Once the upload is complete, the **Edit Information & Diary** screen appears. (Fig. 3)

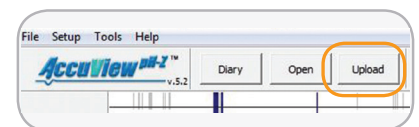


Fig. 3

4. If more than one protocol is available for the Bravo pH capsule, select the desired one from the **Protocol** list. (Fig. 4)

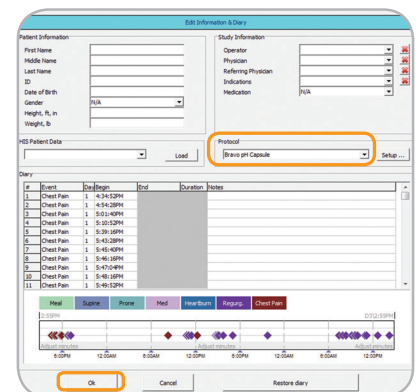


Fig. 4

5. Click **OK**. (Fig. 4)

6. Unplug the **USB cable**. You may proceed with editing the study. Be sure to save the study in the AccuView analysis software.



*Expanding the scope of GI*

[www.givenimaging.com](http://www.givenimaging.com)

Copyright ©2001-2013 Given Imaging Ltd. GIVEN, GIVEN & Design, PILLCAM, PILLCAM & Logo, RAPID, RAPID ACCESS, BRAVO, BRAVO PH SYSTEM, DIGITRAPPER, MANOSCAN, MANOSHIELD, MANOVUEW, GASTROTRAC, GEROFLEX, VERSAFLEX, ACCUTRAC, ACCUVIEW, POLYGRAF ID, SMARTPILL, MOTILIGI, SMARTBAR, and THE MEASURE OF GI HEALTH are Trademarks and/or Registered Trademarks of Given Imaging Ltd., its subsidiaries and/or affiliates in the United States and/or other countries. All other company or product names are the trademarks or registered trademarks of their respective holders. All rights not expressly granted are reserved.

5-001-737