

Indigo Carmine

STAINING

INDIGO CARMINE 1% - 10 ML


Accurately reveals the abnormalities of the mucosal relief (ulceration, fistula, unevenness) and thus marks the tumour margins when the limits are vague.

DETECT
NON POLYPOID
LESIONS OF
SMALL SIZE

INCREASE THE RELIEF
OF THE MUCOSA

DEFINE
THE TYPE
OF LESION

SPRAY CATHETER

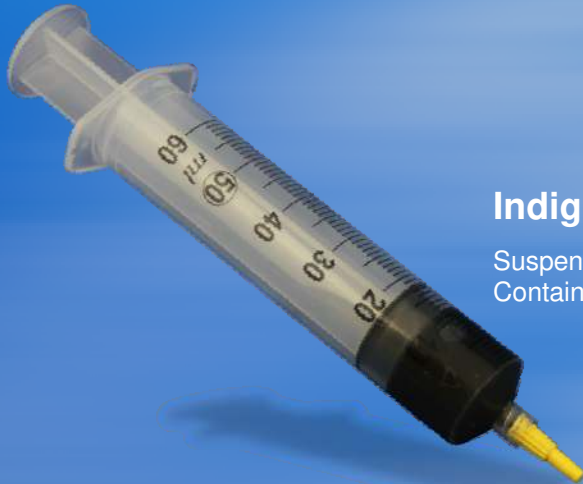

PRESENTATION

Indigo Life SYRINGE (x10)

Suspension of indigo carmine in water.
Contains 10 ml in a 60 ml syringe.

USE

Source: SFED Website: www.sfed.org
Photos by Jean-Marc CANARD

Colonic mucosa
stained with indigo
carmine



Villous layer after
staining



Flat adenoma of the
colon after staining






● DILUTION

Recommended dilution in digestive endoscopy: 0,2% to 0,4%. To achieve the dilution, add saline solution.

● OPERATING PROCEDURE

Spray the indigo carmine solution on the area you need to examine. Repeat if necessary. If the staining is too strong, it can be faded by rinsing.

REFERENCES

PRODUCT CODE	DESIGNATION – TECHNICAL FEATURES	CONTENT
INDIGO	Indigo carmine - 10 ml Box of 10 syringes – Single use	
99-01-01-31-0	Spray catheter Length of catheter : 230 cm Catheter diameter: 2.4 mm - Sterile – Single use	
KIT INDIGO	KIT Indigo Life 10 syringes of Indigo Life + 10 spray catheters	

DISTRIBUTOR
DIAGMED HEALTHCARE
Thirsk
North Yorkshire, YO7 3TD
Tel: 01845 526660
Fax: 01845 522199
www.diagmed.co.uk