

HaemoCer™ PLUS

DAPI application technique

1



Dry

Remove excess pooled blood by suction or gauze

2



Apply

Apply HaemoCer™ PLUS immediately and extensively to the wound site

3



Press

Apply gentle compression over the wound using dry gauze - for up to 2 minutes in the case of profuse bleeding

4



Irrigate

Irrigate thoroughly prior to gently removing the gauze to ensure the gelled matrix remains at the wound site

Product overview HaemoCer™ PLUS

HFP201	1g	5 pcs	HFZ101	440 mm	5 pcs
HFP202	2g	5 pcs	Universal Applicator		
HFP203	3g	5 pcs	HFZ201	180 mm	5 pcs
HFP205	5g	5 pcs	FlexApplicator		

BioCer

BioCer Entwicklungs-GmbH

Ludwig-Thoma-Str. 36c · 95447 Bayreuth · Germany
 Tel. +49 (0)921 78 77 70 0 · Fax. +49 (0)921 78 77 70 79
 info@biocer-gmbh.de · www.biocer-gmbh.de

