# Assisted Capsule Endoscopy

AdvanCE™ Case Report Series • Report 3



"Very simple to endoscopically place the capsule under conscious sedation into the stomach or duodenum."

Robert A. Muggia, M.D. Chief of GI Winchester Hospital Winchester, MA

### **Procedure**

• EGD with video capsule placement

## **Indications for Capsule Endoscopy**

• Idiopathic Iron Deficiency Anemia

#### Indications for AdvanCE™ Device

Phobic aversion to pill swallowing


## **Assisted Capsule Endoscopy**

After adequate conscious sedation was achieved, the scope was advanced under direct visualization to the 3rd portion of the duodenum for baseline evaluation. No source of anemia determined; video capsule placement elected. The scope was withdrawn and the AdvanCE<sup>TM</sup> device loaded for endoscopic video capsule delivery. Deployment of the video capsule into the 2nd portion of the duodenum was successful.







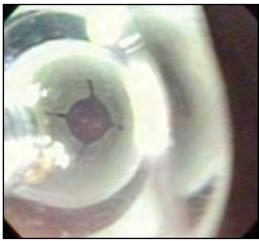

2. AdvanCE device's empty capsule cup after successful video capsule deployment



listening... and delivering solutions

5976 Heisley Road Mentor OH 44060

phone 440 / 639.4494

fax 440 / 639.4495

customer service 800 / 769.8226

www.usendoscopy.com

This case report is not intended to provide device or clinical instruction or intended to be a substitute for the Instructions For Use (IFU) which are packaged with the device. Please consult the IFU prior to using the AdvanCE™ capsule endoscope delivery device. AdvanCE™ is a trademark of US Endoscopy, patent pending.