## AdvanCE® – capsule endoscope delivery device

AdvanCE® – Système d'introduction de capsule endoscopique  
AdvanCE® – Endoskopische Videokapsel-Absetzvorrichtung  
AdvanCE® – dispositivo di posizionamento per capsula endoscopica  
AdvanCE® – dispositivo para la colocación endoscópica de cápsulas  
AdvanCE® – dispositivo de colocação de cápsula endoscópica  
AdvanCE® – anordning til indføring af kapsel ved endoskopi  
AdvanCE® – inbrenginstrument voor capsule-endoscoop  
AdvanCE® – kapsül endoskop iletim cihazı  
AdvanCE® – 캡슐 내시경 전달 장치

## Reorder No. 00711144

Référence de commande n° 00711144  
Nachbestell-Nr. 00711144  
N. di riordino 00711144  
N.º de pedido 00711144  
N.º de encomenda 00711144  
Genbestellingsnr. 00711144  
Nabestelnr. 00711144  
Sipariş No: 00711144  
주문 번호 00711144

---

## INSTRUCTIONS FOR USE

Mode d'emploi  
Gebrauchsanleitung  
Istruzioni per l'uso  
Instrucciones de uso  
Instruções de utilização  
Brugsanvisning  
Gebruiksaanwijzing  
Kullanım Talimatları  
사용 설명서

---

*listening... and delivering solutions\**

5976 Heisley Road  
Mentor, OH 44060 USA  
**phone** +1 440 / 639.4494  
**fax** +1 440 / 639.4495  
**email** global@usendoscopy.com  
[www.usendoscopy.com](http://www.usendoscopy.com)

**This device is not made with natural rubber latex.**

**Intended Use:**

- The disposable AdvanCE® capsule endoscope delivery device is a 2.5mm single sheathed device indicated for transendoscopic delivery of Capsules (with the dimensions of 10.5mm - 11.5mm in diameter and 23.5mm - 26.5mm in length), to the stomach or duodenum. This device is intended for patients who are either unable to swallow the capsule, or unable to pass the capsule beyond the pylorus in sufficient time to complete the desired diagnostic evaluation.

Description	Product Number	Sheath Diameter	Sheath Length	Acceptable Capsule Sizes
Advance® Capsule Endoscope Delivery Device	00711144	2.5mm	180cm	Diameter: 10.5mm - 11.5mm Length: 23.5mm - 26.5mm

Capsule Cup - Outer Diameter (OD)	0.50 inches / 12.7mm
Capsule Cup - Inner Diameter (ID)	0.46 inches / 11.7mm

**Warnings and Precautions:**

- Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
- Consult the medical literature relative to complications, hazards and techniques prior to the performance of any endoscopic procedure.
- For use of the video capsule, consult the Information for Use document from the video capsule manufacturer.
- Transendoscopic capsule placement requires skill and experience in endoscopic esophageal intubation **with an accessory device seated at the distal tip of the endoscope**. The use of this device is not recommended if the clinician lacks the required experience and proficiency.
- This device is compatible with an endoscope that has an accessory channel of 2.8mm or greater.**
- Do not attempt to reuse, reprocess, refurbish, remanufacture or resterilize this device. US Endoscopy did not design this device nor is it intended to be reused, reprocessed, refurbished, remanufactured, or resterilized. Performing such activities on this disposable medical device presents a safety risk to patients (i.e. compromised device integrity, cross-contamination, infection).

**Contraindications:**

- Those specific to any GI condition that may hinder or prevent the endoscopic placement of the capsule.
- The Given PillCam® SmartPill Motility capsule is not compatible with the AdvanCE® Capsule Endoscope Delivery Device.**

**Prior to Use Instructions:**

- Perform baseline EGD.
- Assess if the patient is a candidate for capsule deployment (see contraindications).
- Decompress the stomach upon completion of the baseline EGD (this can minimize gastric looping on the repeat EGD for capsule delivery).
- Prior to clinical use, you should familiarize yourself with the AdvanCE® capsule endoscope delivery device.
- Read this "Instructions for Use" and review the attached photographs.
- Open and inspect the device for shipping or handling damage. If damage is evident (e.g. bends, kinks, cracks, misshaped cup), **do not use this device and contact your local product specialist**.

**Instructions for Use:**

- Open the larger poly bag, remove the paper tape restraint from the coiled device, and gently uncoil the catheter assembly.
  - Remove the small capsule cup bag by sliding it over the end of the catheter.
  - Hold the proximal end in one hand and the distal sheath in the opposite hand, and drape in a "U" shape.
  - Open and close the finger rings on the handle two (2) times to make sure it moves smoothly. (Moving the finger rings away from the thumb ring is open, moving the finger rings towards the thumb ring is closed).
    - Observe that the deployment cable moves in and out.
- Note: The finger rings do not advance the full length of the handle shaft.**
- If this unit does not function properly, **do not use this product and please contact your local product specialist**.

6. Place the scope in a straight position prior to inserting the AdvanCE® capsule endoscope delivery device catheter.
7. Hold the finger rings on the AdvanCE® device handle in a closed position (see Fig. 1) to ensure that the deployment cable is maintained within the catheter.
8. Pass the catheter through the endoscope's accessory channel (See Fig. 2) using short strokes (1" -1.5" length is recommended to avoid sheath kinking).
9. Observe that the catheter assembly exits the distal end of the endoscope.
10. Remove capsule cup from the small bag.
11. Hold the white portion of the capsule cup between thumb and forefinger, with clear end facing out.
12. Attach the clear end of the capsule cup onto the threaded end of the catheter by turning the capsule cup in a clockwise direction (see Fig 3).
13. Tighten capsule cup until it stops and the catheter begins to turn.
14. Pull gently on the capsule cup to confirm proper connection.
15. Hold the finger ring handle in a closed position (see Fig. 1).
16. Push the video capsule into the capsule cup with the optical dome of the video capsule facing out (see Fig. 4).
17. Listen for an audible click (This indicates that the capsule is appropriately seated within the capsule cup).

**Caution: Do not use the device if you experience any of the following:**

- a. Failure to hear an audible click
- b. Capsule falls out of capsule cup

18. Seat the capsule cup against the endoscope's distal tip by pulling the proximal end of the catheter (see Fig. 5).
19. Lubricate the outer surface of the capsule cup and endoscope with water soluble lubricant.

**Note: Do not cover the endoscope lens or video capsule with lubricant**

20. Hold the finger rings on the AdvanCE® capsule endoscope delivery device handle in a closed position (see Fig. 1).
21. Maintain tension on the catheter during intubation to keep the capsule cup against the end of the endoscope.
22. Advance the capsule cup and endoscope into the stomach or duodenum.

**Note: Patient condition may indicate the best location for capsule deployment.**

23. Deploy the capsule by following these steps:

- a. Open the finger ring handle **briskly** until it stops (see Fig. 6).
- b. Confirm capsule delivery endoscopically by looking through the clear capsule cup and seeing the cable and the empty capsule cup.

**Note: Advancing the capsule cup from the distal tip of the endoscope may enhance the luminal field of view.**

24. If the capsule does not fully deploy:

- a. Close the finger ring handle.
- b. Slightly alter the position/angulation of the endoscope.
- c. Straighten the handle and catheter
- d. Repeat step 23.


25. After capsule delivery, close the finger ring handle until the deployment cable is retracted into the catheter.

26. Maintain tension on the catheter to keep the capsule cup seated against the endoscope's distal tip.

27. Remove the endoscope from the patient.

28. Remove the device from the endoscope:

- a. Hold the finger ring handle in a closed position.
- b. Loosen the capsule cup until it can be removed from the distal end of the catheter.
- c. Withdraw the device from the endoscope's accessory channel.
- d. Dispose of device as per institutional guidelines.

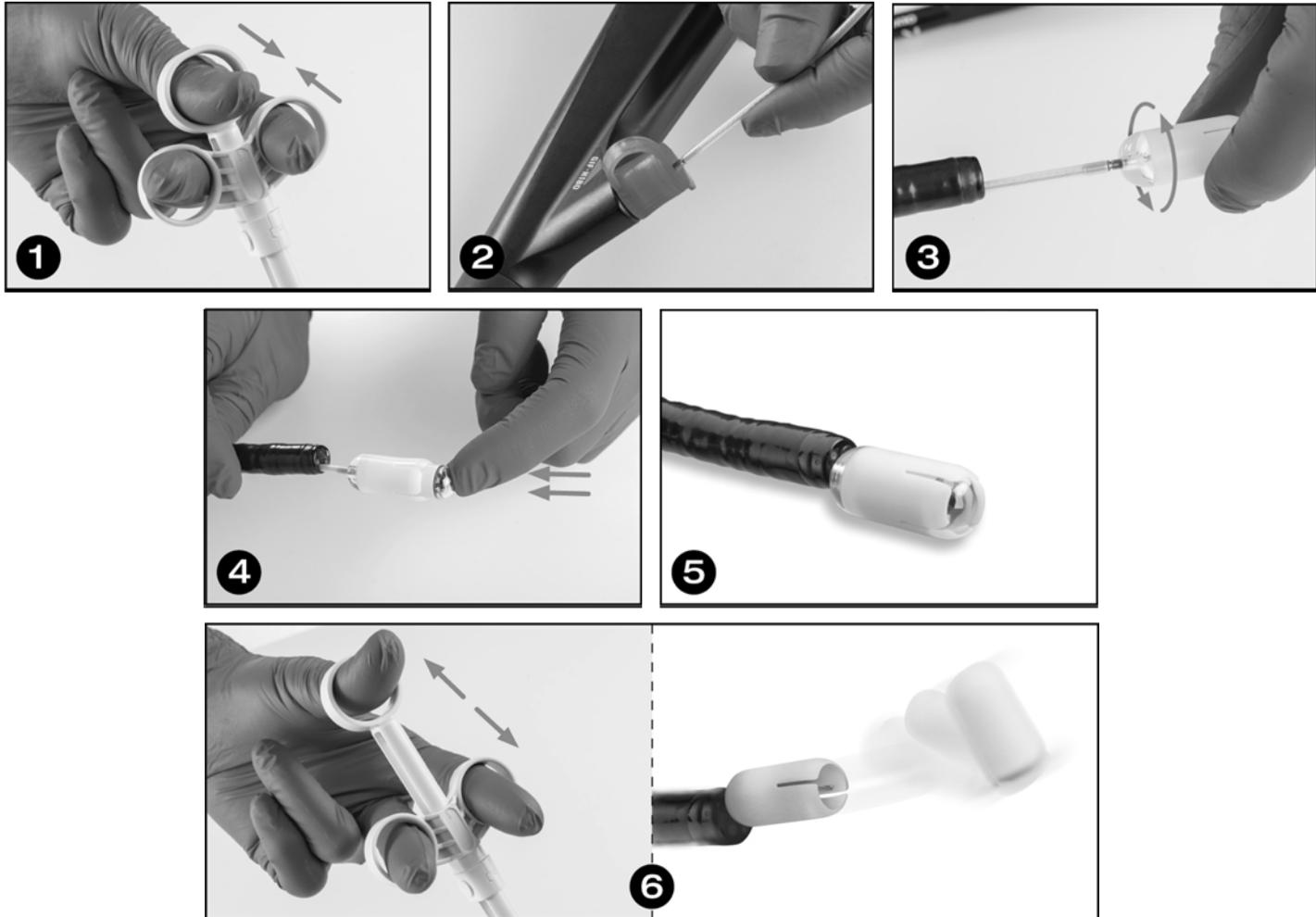**Product Disposal:**

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

**Issued Date:** January 2018

**Warning:**

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy

US Endoscopy, a wholly owned subsidiary of STERIS Corporation.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont des marques déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions®, das US Endoscopy listening...and delivering solutions-Design und alle mit ® oder ™ bezeichneten Marken sind beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragene Marken sind Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions-design og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions®, het ontwerp Listening...and delivering solutions van US Endoscopy en alle aanduidingen met ® of ™ zijn gedeponiert bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™으로 표시된 모든 상표는 미국 특허청(U.S. Patent and Trademark Office)에 등록되어 있거나 US Endoscopy가 소유하고 있는 상표입니다.